How Do I Turn Off My ScanVision?

Last Modified on 06/26/2018 1:45 pm PDT

Table of Contents

- Summary
- The Power Switch and Power Button
- Adjusting Power Saving Mode
- Quick Tips

Summary

The ScanVision has both a **Start Button** and **Power Switch**. This article will demonstrate the location and purpose for each, as well as general tips for using each.

The Power Switch and Start Button

The Power Switch is located on the right side of the device and can be set to On or Off. The Power Switch should only be changed when the device is transported in a bag, case, box, etc. or being shipped.


After each exam the devices will turn off automatically. However, you can also turn the device off by holding down the **Start Button** for 8 seconds. The Start Button is either white or black on the top of the device.

Once the **Power Switch** is set to "**On"** the device will start each time you push the **Start Button**.

Adjusting the Power Saving Mode

You can adjust the amount of time your devices will stay on before shutting off.


1. Open Setup, go down to Settings, and select Devices.

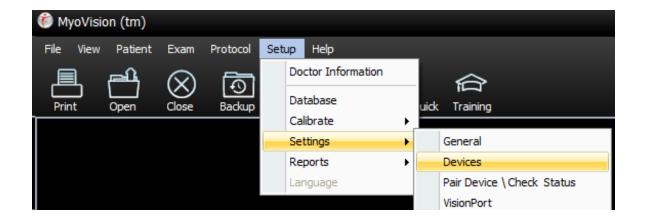

2. Select ScanVision from the options on the left.



3. By adjusting the Battery Saving Mode setting you can set how long the device will stay on without being used before turning off.

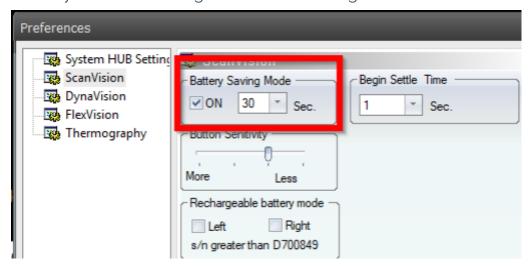

Quick Tips

The **Power Switch** on the side of the device should be adjusted similar to how you would flip a light switch on/off. Using a paper clip against the side of the switch,

adjust the **Power Switch** by pushing the paper clip to the **SIDE**. **DO NOT push down** on the top of the power switch as this may damage the device and prevent it from working properly.

