

# How to View Exams Side-by-Side

Last Modified on 09/11/2019 1:16 pm PDT


## Summary

The purpose of this article is to demonstrate how to open exams side-by-side in the MyoVision Software. This is a great way to compare to the ideal results or to track progress against previous exams.


1. [DynaROM exams](#)
2. [Static sEMG exams](#)

## DynaROM Exams

1. Click on "Setup" at the top menu, go down to "Settings" and select "General."

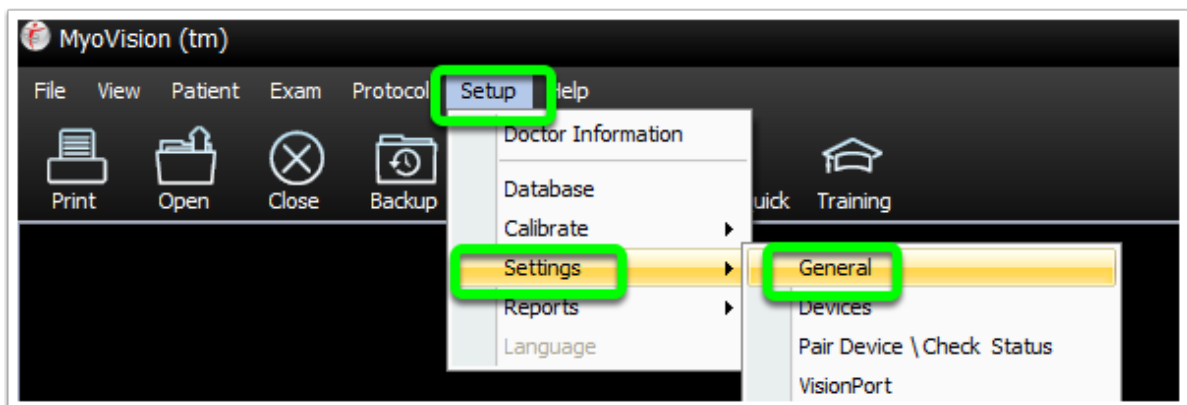


2. Click the box next the "Show add/replace exam prompt" so that it is checked.

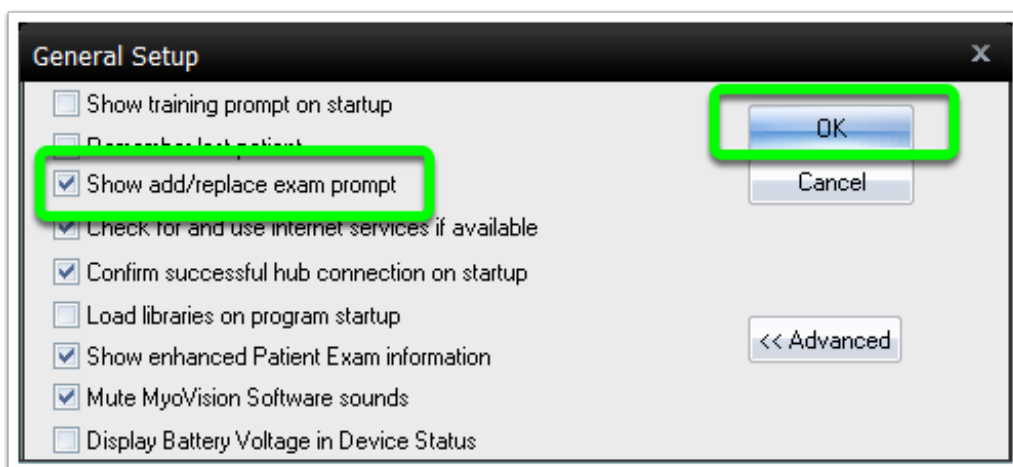


3. While you already have an exam open, click the "Open" button at the top of


the software and select the exam you want to open.

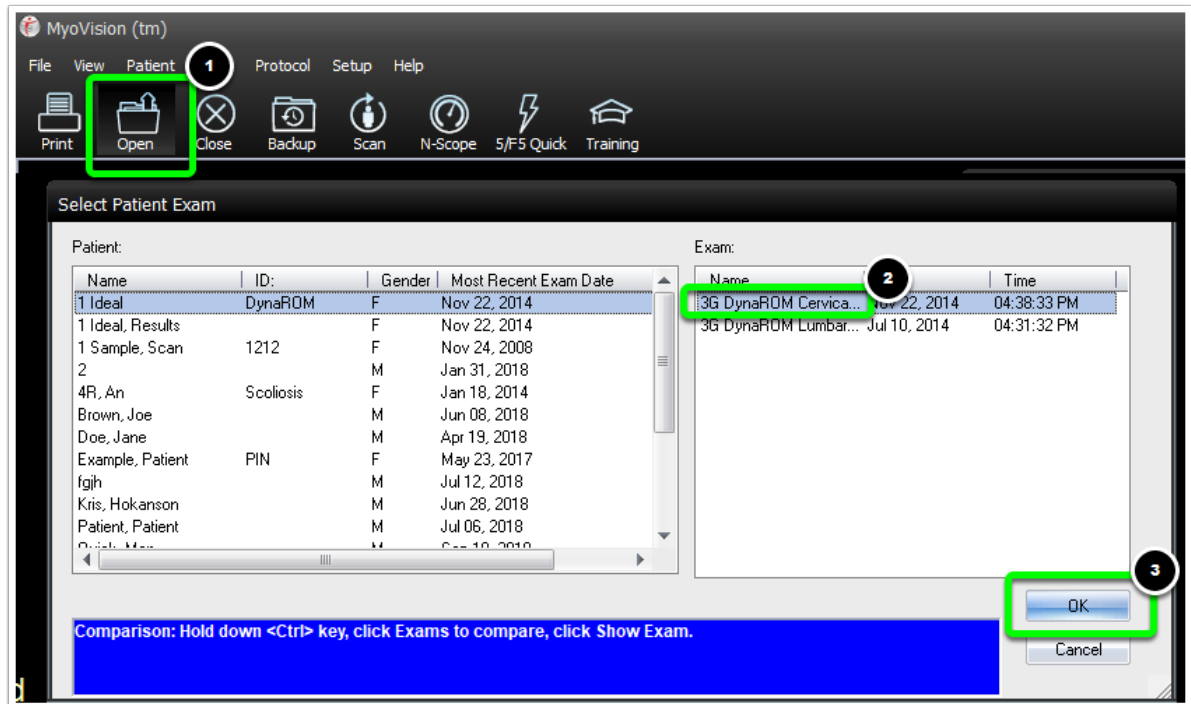

4. When prompted select the "Add to open exams" option. Then, click "OK."

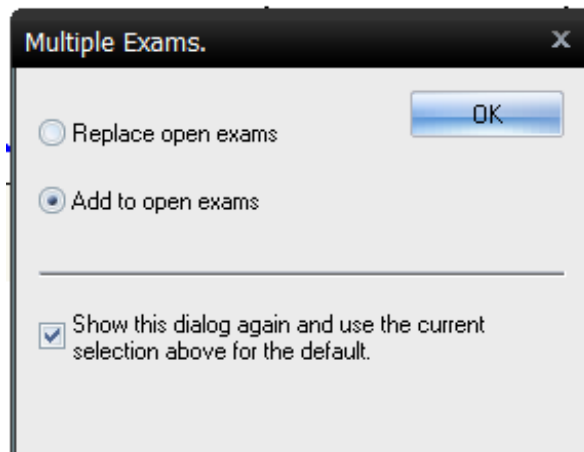


5. The second exam will now open right next to the first exam.

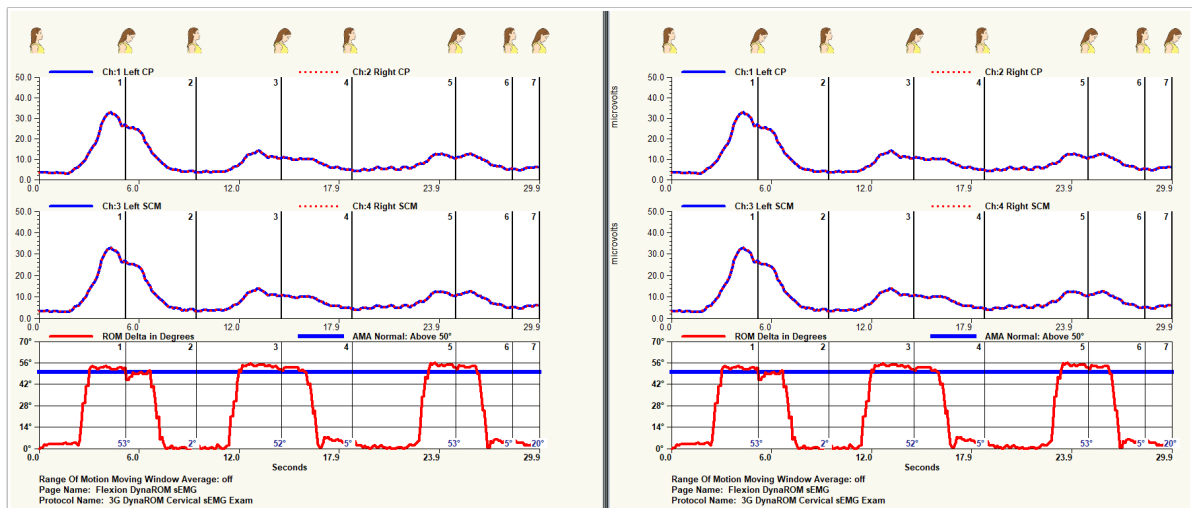


## Static sEMG Exams

In this section you will learn how to view patient exams side-by-side without seeing the "Goal".

1. Click on "Setup" menu, go down to "Settings" and select "Static."

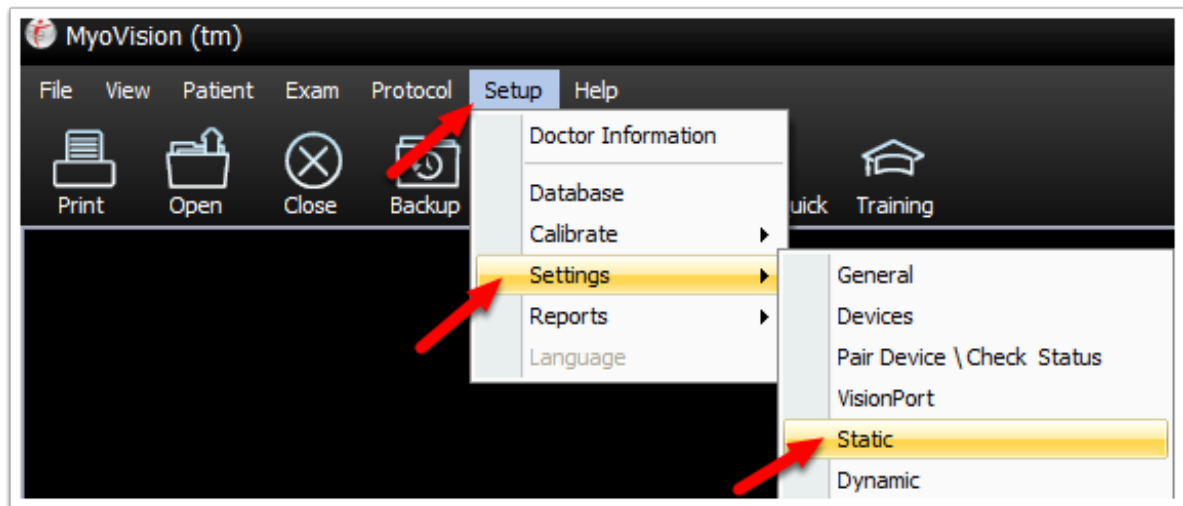

2. A window will open, click the check box labelled "Show Goal" to remove the check mark. Then, click "OK" to save your settings.

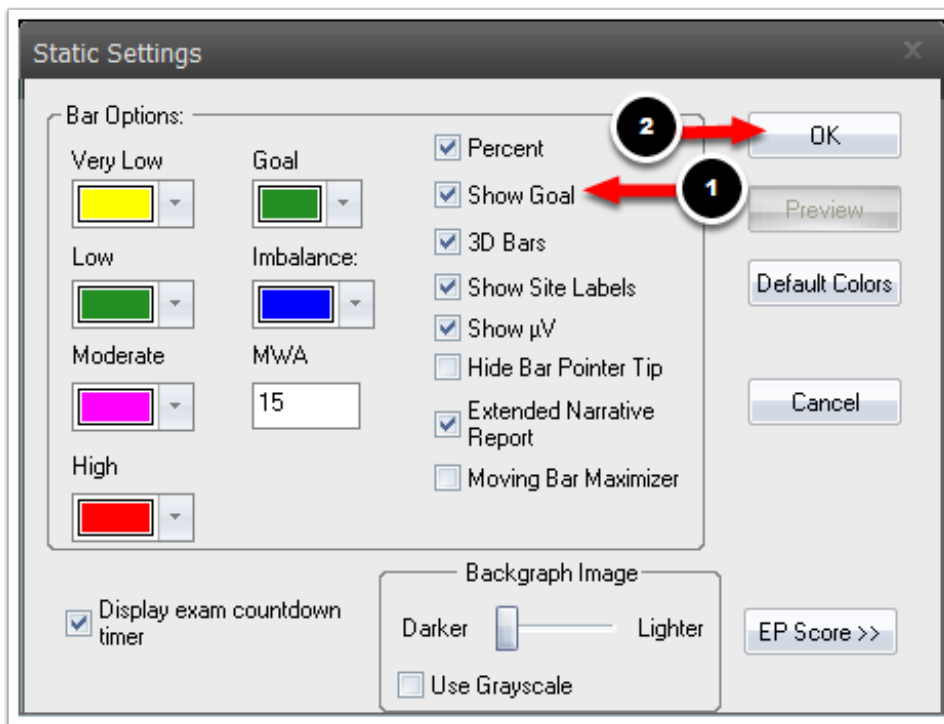

- Now when you view patient exams, you will be able to view them side-by-side without viewing the "Goal" exam.

