# How to Edit Drop-Down Options in Interpretation Reports

Last Modified on 01/15/2018 2:50 pm PST

This article will show you how to edit the content options for your interpretation reports. Some fields are locked and you cannot edit them directly, you must go through Microsoft Word's Developer tool to edit the content's properties.

#### Before You Begin:

- Have your report open and ready to edit
- You can edit a report you created or the template itself

### Editing a Created Report vs. the Template

If you only have one small change you would like to make for a specific person it is best to just edit the report you created.

If you would like to add drop down options across all of your reports, you should edit the Template file located in C:/WinScang8/Template

#### 1. Turn on Developer Mode

Print Share	Protect Document *	General Display Proofing	General options for working with Word.
Export Close	Inspect Document Before publishing this file, be aware t	Save Language Advanced	<ul> <li>✓ Show Mini Toolbar on selection <sup>①</sup></li> <li>✓ Enable Live Preview <sup>③</sup></li> <li>✓ Update document content while dragging <sup>③</sup></li> <li>ScreenTip style: Show feature descriptions in ScreenTi</li> </ul>
Account Feedback Options	Check for Issues * E Document properties, author E Footers Custom XML data Content that people with disabil	Customize Ribbon Quick Access Toolbar Add-ins Trust Center	Personalize your copy of Microsoft Office       User name:     EA       Initials:     E       Qlways use these values regardless of sign in to Office
	Find	Add >> Remove Customize the Main Tabs Customize the Main Tabs Customize the Main Tabs Customize the Main Tabs Customize the Customize the Custom	v log Post) g und Removal ioard graph s ig tes

You need access to Microsoft Word's Developer tools

If you do not see the **Developer** tab:

- 1. Go to File > Options
- 2. Select Customize Ribbon
- 3. Check mark Developer in the right column
- 4. Click OK

## 2. Content Properties

#### **Cervical Left Lateral Flexion**

- 1. Level of Activity of Opposite Side (Muscle Guarding / Co-Contraction): Choose an item.
- Co-Contraction
- Consistency across Trials: Choose an item.
- 3. Fibrillation of muscles in motion: Choose an item.
- 4. Range of Motion in Degrees. AMA: 30<sup>d</sup> Patient: 26<sup>o</sup>
- 5. Quality of motion: Choose an item.



- Select the content field you wish to edit
- Open the Developer Tab
- Select Properties

#### 3. Edit Drop-Down List Properties

eneral							
Title: Co-Contraction							
Tag: Co-Contraction							
Show as: Bounding Box 🗸		Ch	In Defe	WH Dava and	unh Ennt		
Color: 🕭 🔻		Sty			aph Font 🗸	-	~
Use a style to format text typed into th	ne empty control	Modi	fy Choice	e		?	×
Style: Default Paragraph Font			Display <u>N</u> ame: L <u>V</u> alue:	This is a custom edit			
Ha New Style				This is a (	custom edit: I	E <mark>dit this</mark>	field
Remove content control when content	ts are edited			Г	OK	C	ancel
ocking		Drop-Do	own <u>L</u> ist P	Properties			
	<u>A</u> dd	Drop-Do	own <u>L</u> ist F	Properties			
ocking Content control cannot be <u>d</u> eleted Contents cannot be <u>e</u> dited rop-Down <u>List Properties Display Name Choose an item.</u>	<u>A</u> dd	Drop-Do	own <u>L</u> ist F	Properties			
Ocking         Content control cannot be deleted         Contents cannot be edited         rop-Down List Properties         Display Name         Choose an item.         Normal: The patient shows no bracing         Moderate: Left and right sides fire simu		Drop-Do	own <u>L</u> ist F	Properties			
Ocking         Content control cannot be deleted         Contents cannot be edited         rop-Down List Properties         Display Name         Choose an item.         Normal: The patient shows no bracing	<u>A</u> dd <u>M</u> odify	Drop-Do	own <u>L</u> ist F	Properties			
Ocking         Content control cannot be deleted         Contents cannot be edited         rop-Down List Properties         Display Name         Choose an item.         Normal: The patient shows no bracing         Moderate: Left and right sides fire simu	<u>A</u> dd <u>M</u> odify Rem <u>o</u> ve	Drop-Do	own <u>L</u> ist F	Properties			

- Choose the option you would like to edit.
- Click on Modify

- Editing the **Display Name** will change the **Value** name, but not the other way around.
- Make sure you edit the Value field
- Click OK after making your changes and OK out of the properties window.

Repeat as necessary to edit different content fields.