

How to Calibrate the FlexVision

Last Modified on 02/19/2018 4:30 pm PST


If you have noticed that your inclinometer readings may be a little off, it may need to be calibrated.

Before you begin:

- Connect your FlexVision system to your computer
- Launch the MyoVision Software

1. Menu Navigation

To access the calibration settings for inclinometers go to Setup > Calibrate > ROM

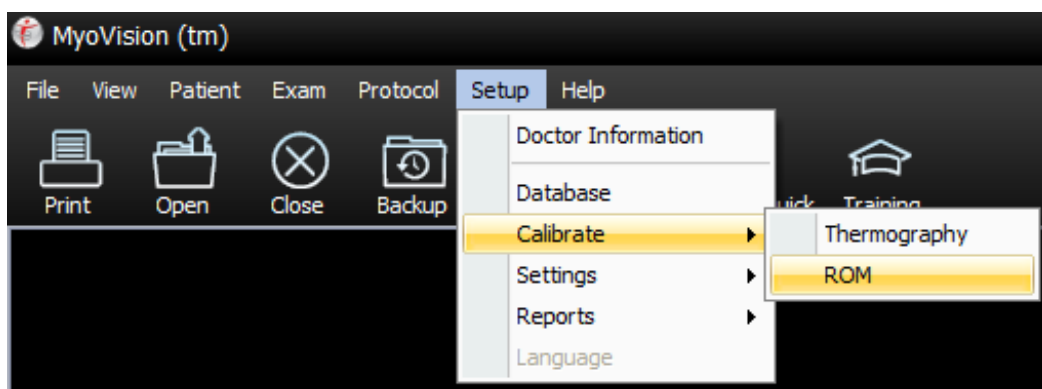


2. Calibration

The calibratoin window will have instructions for you to follow.

Important: Only calibrate one device at a time and ensure the surface is as flat as possible.

Begin calibration by pressing and releasing **one** of the start buttons on the device. The label will change from Pending > Hold Steady > Calibrated when finished calibrating the device

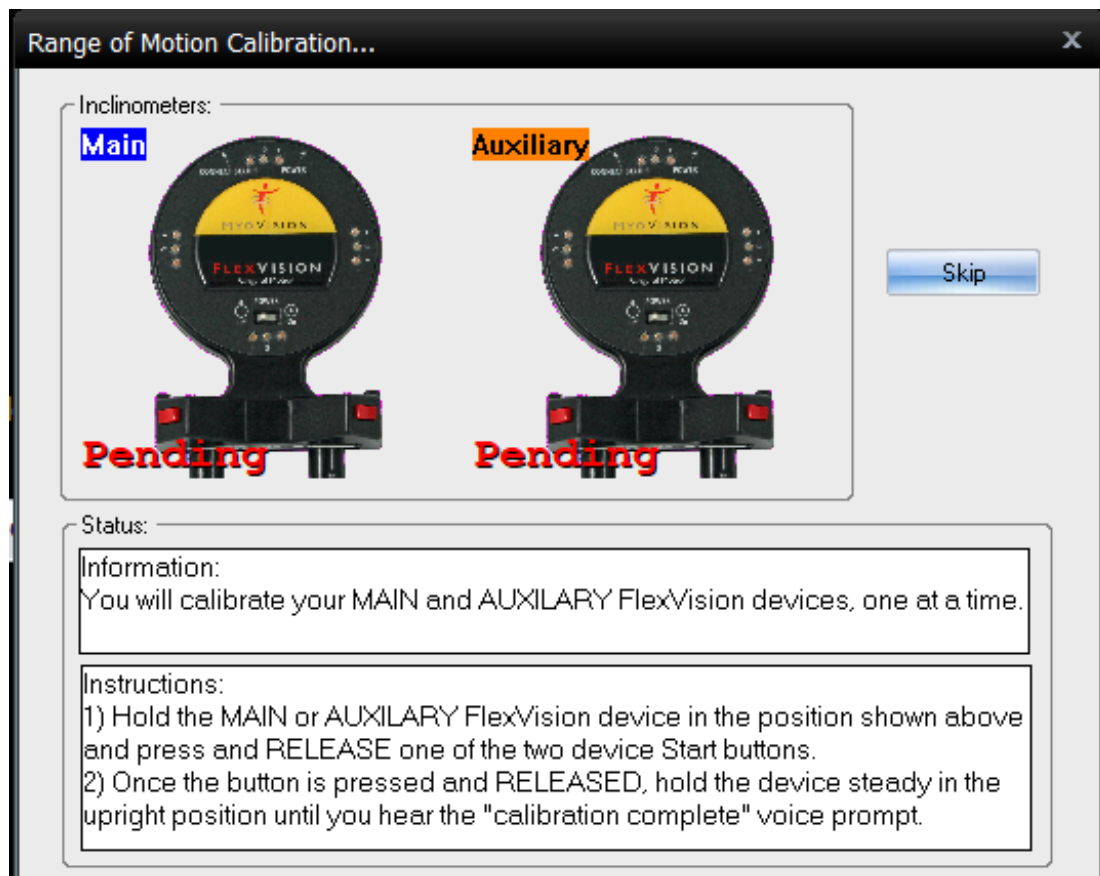

3. Finished


Both devices will say Calibrated! when calibration has completed.

Click on Finish to close out of the screen.

Range of Motion Calibration... X


Inclinometers:

Main



Calibrated!

Auxiliary



Calibrated!

Status:

Information:
Both Main and Auxiliary FlexVision devices calibrated.

Instructions:
Press and release one of the red START buttons on the device or press the [Finish] button to the right to continue.