# Unable to Download Software

Last Modified on 07/10/2018 1:52 pm PDT

## Table of Contents

- Summary
- Use Internet Explorer
- Further Questions and Readings

#### **Summary**

This article will take you through steps to troubleshoot issues downloading the MyoVision software.

#### Internet Explorer

If you are getting a blank page when trying to visit the download page for the MyoVision software or when clicking on the Download Now button, the first step is to try accessing the page using Internet Explorer.

To open Internet Explorer in Windows 10:

1. Open the search bar, search for Internet Explorer. Then click to launch it.



2. Go to myovision.com/download and click the Download Now button.



 After clicking Download Now you will be prompted to save or run the "MVSW\_ATLAS\_S" file. This file is the MyoVision software install file. Select Save.

Do you want to run or save MVSW_ATLAS_S.exe (290 MB) from myovision.hostedftp.com?					
	Run	Save	-	Cancel	

4. After the download has finished click the Run option.

The MVSW_ATLAS_S.exe download has completed.	Run	Open folder	View downloads	×

From here the installation of the MyoVision software will begin automatically.

### **Further Questions and Readings**

If you are still having trouble getting the page to show up you will want to contact your Office's IT staff to check if the website is being blocked by your Network.

MyoVision Technical Support is available Monday to Friday 9 am to 5 pm Pacfific Standard Time,

Phone: (800) 969 - 6961 Option 2. Email: support@myovision.com