

How To Merge Multiple Databases

Last Modified on 07/06/2018 12:48 pm PDT

Summary

This guide will show you merge an older database with a new database, that has patients in it.

Note: If the new database does **not** have new patients in it You can use the guide at the link provided below to backup and restore your old MyoVision database to your new computer.

[Backing Up and Restoring Patient Database](#)

If you are trying to have multiple databases on a network sync their data you will want to read our guide on [Configuring a Network Database in Your Office for MyoVision](#) .

Each step must be fully completed in order to prevent any data loss.

Click on the buttons inside the tabbed menu to view the steps for merging databases:

Step 1: Export Patient
Data

Step 2: Backup the
Database

Step 3: Restore the
Backup

Step 4: Re-Import Your
Exams

Video Tutorials
