


# Change the Primary Language Settings

Last Modified on 04/19/2018 9:09 am PDT

This article will guide you through changing the language setting in the MyoVision software


## 1. Setup > Settings > General

Go to Setup, Settings, and select General from the menu

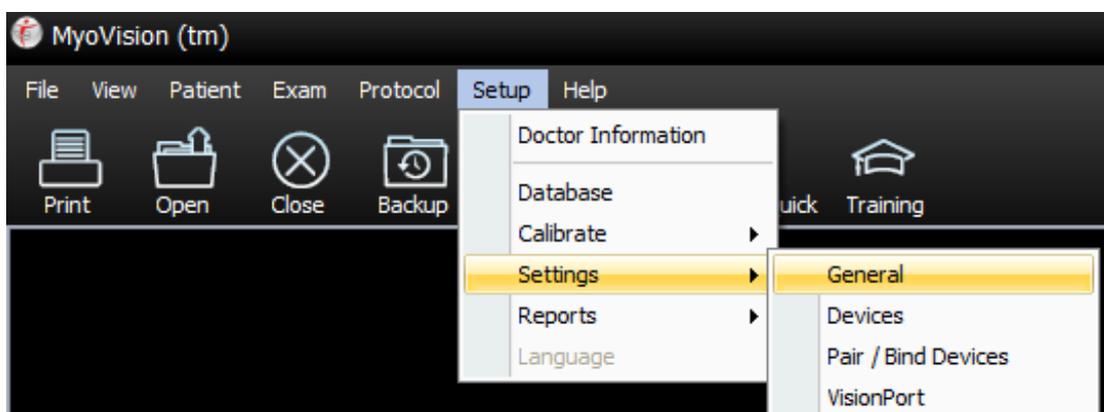


## 2. Advanced

On the General Setup window, select the Advanced button.

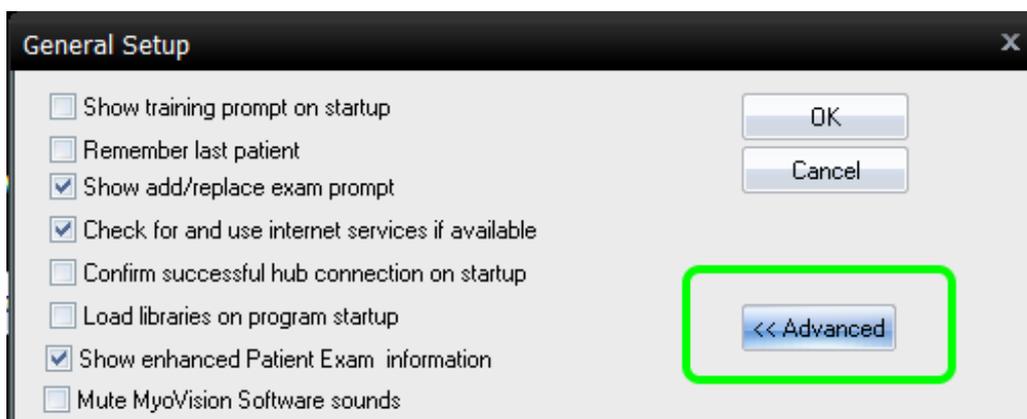


## 3. Enhanced GUI

Remove the check mark next to Enhanced GUI and then click OK

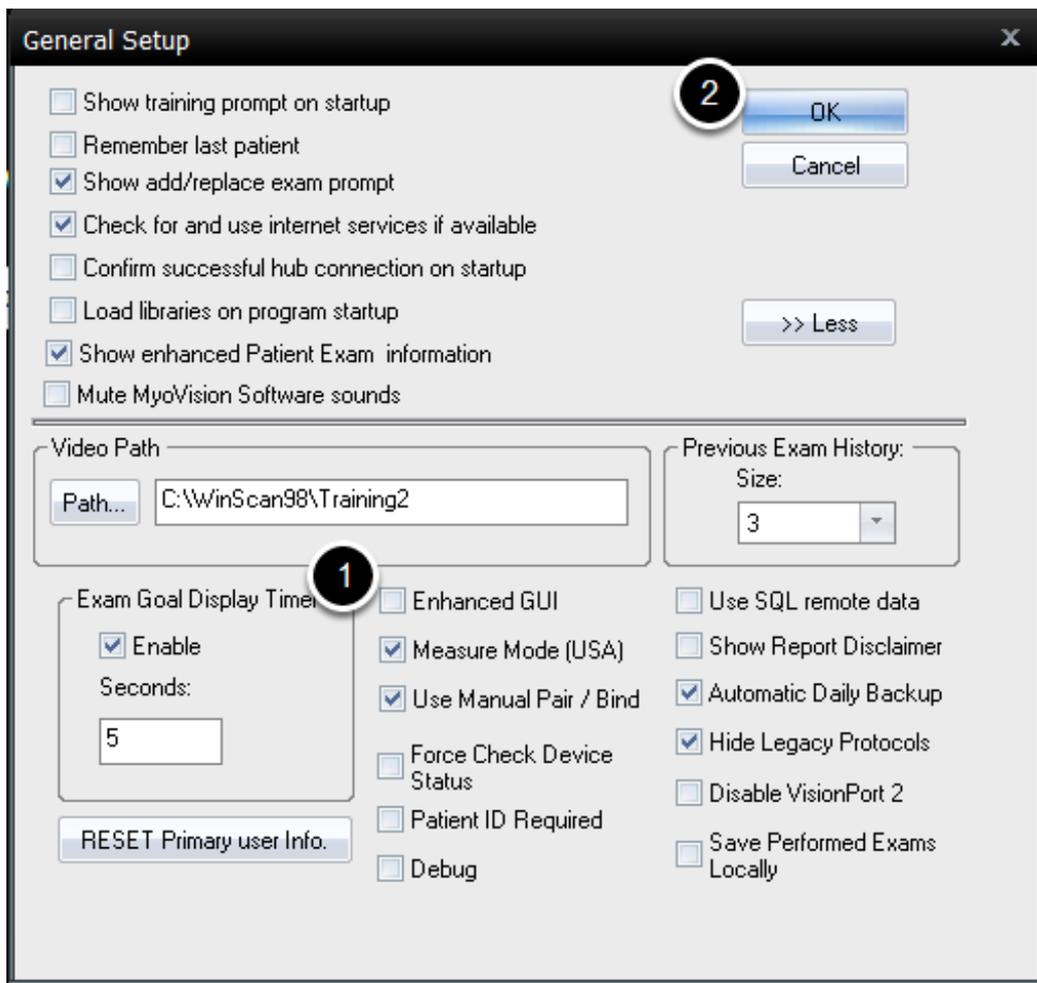


## 4. Restart

You will then be prompted to restart the MyoVision software. Click OK and then close and reopen the software.

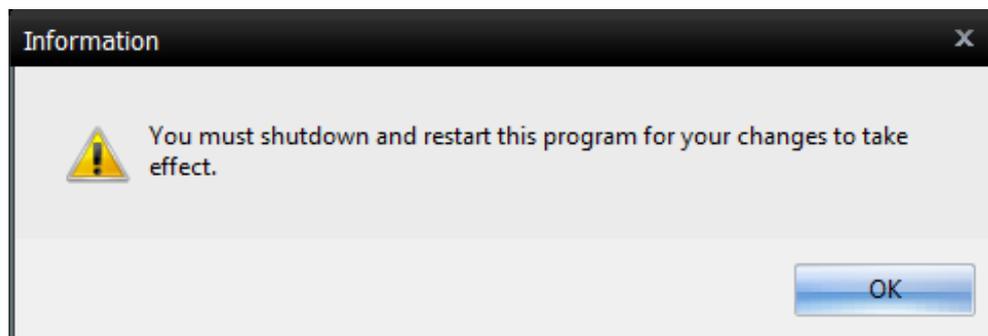


## 5. Setup > Language

Once the software has been opened, go to Setup and then Language and select your desired language.

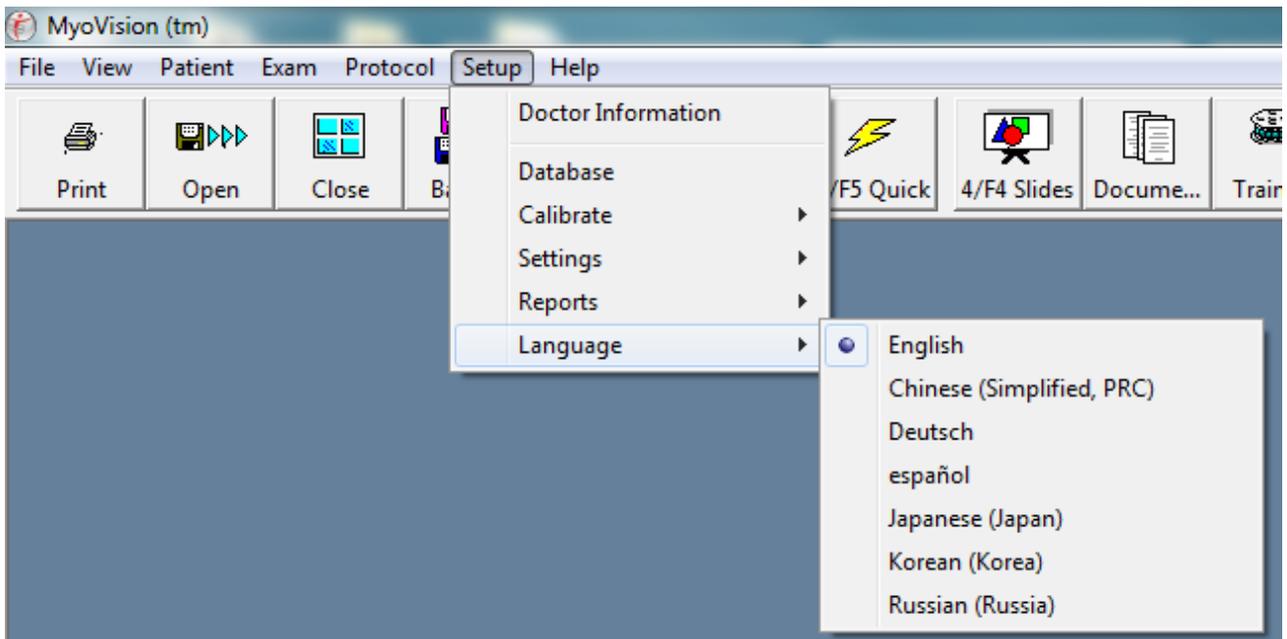


## 6. Don't Restart

Once again you will be prompted to restart the MyoVision Software. Click OK, however, do not restart yet.

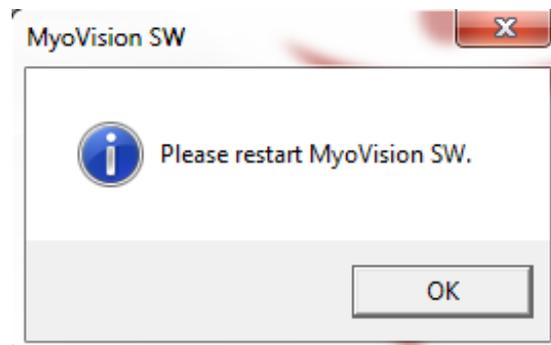


## 7. Setup > Settings > General

Go to Setup, Settings, and select General from the menu

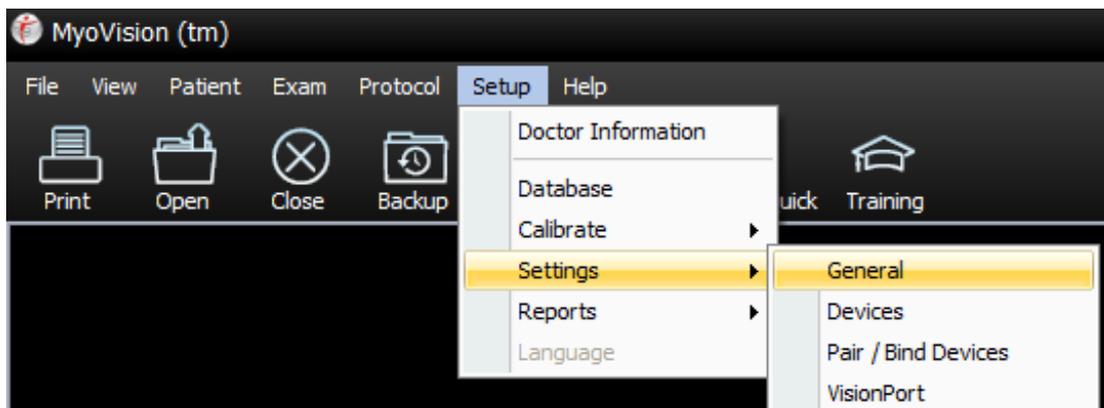


## 8. Advanced

On the General Setup window, select the Advanced button.

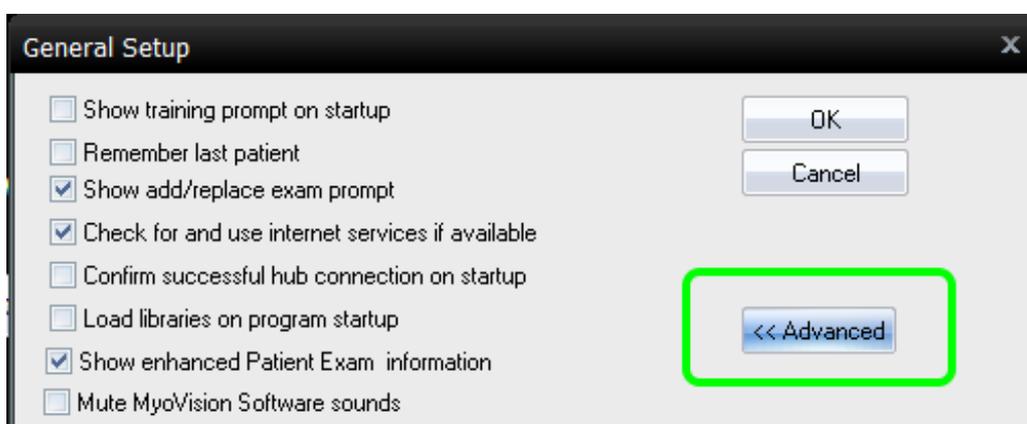


## 9. Enhanced GUI

Place a check mark next to Enhanced GUI and then click OK

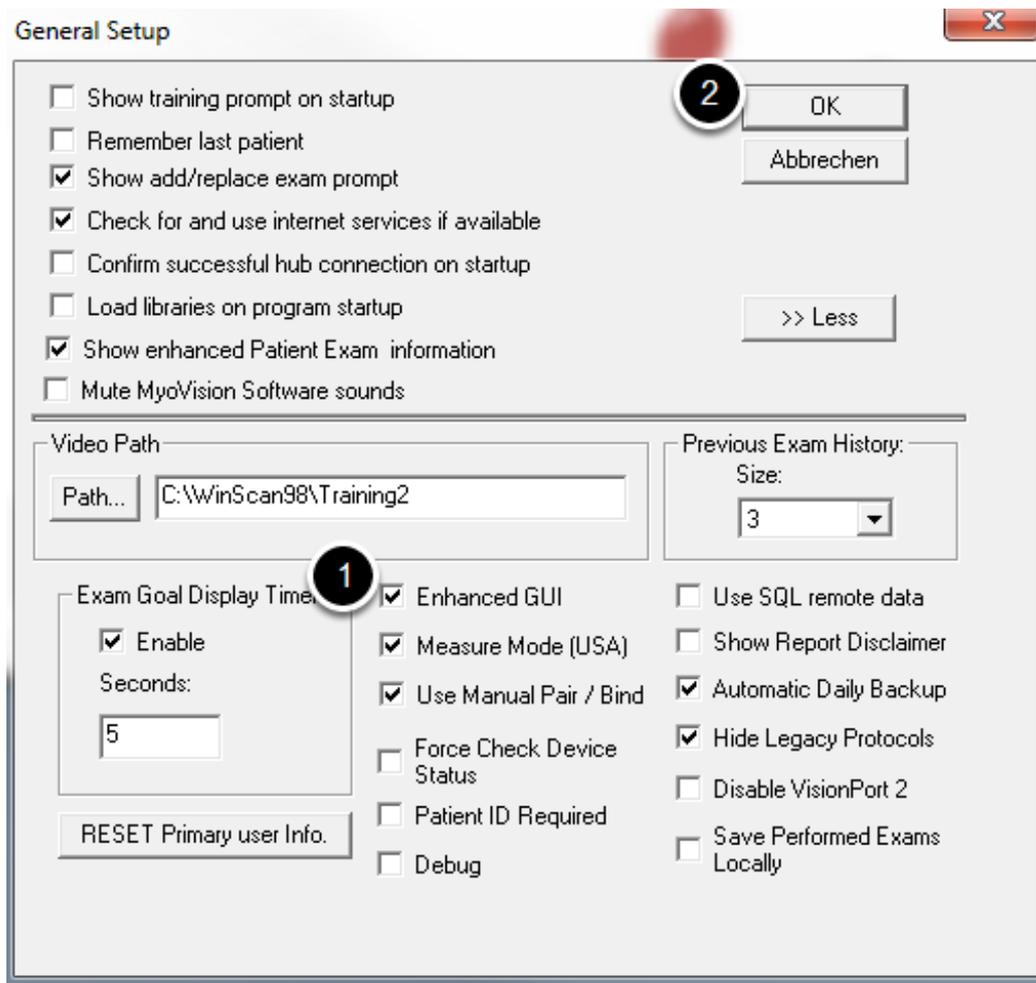

## 10. Restart

You will then be prompted to restart the MyoVision software. Click OK and then close and reopen the software.

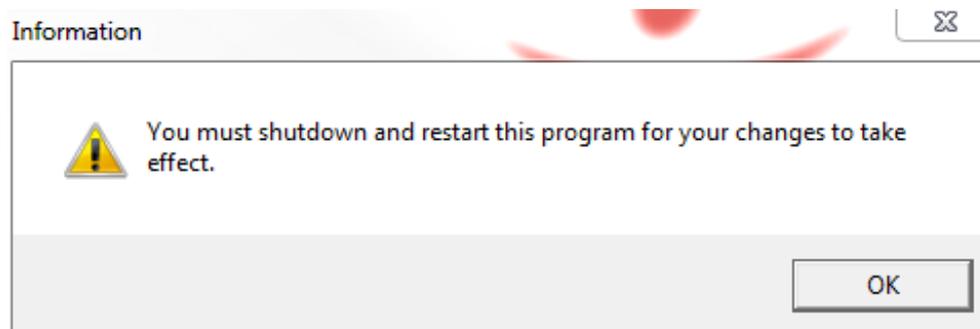

## 11. Finished!

Your software should now have your new language settings selected and back to the enhanced GUI.

