


# How Do I Put Together My DynaROM Belt?

Last Modified on 06/14/2018 12:39 pm PDT


## Summary:

This guide will show you how to assemble the DynaROM belt that patients wear around their waist.

1. Remove the belt from the plastic bag.
2. Take the end of the belt that's cut at a diagonal and pass it up through the slot marked "1" in the image. Then, pass it through the slot marked "2" in the image below.

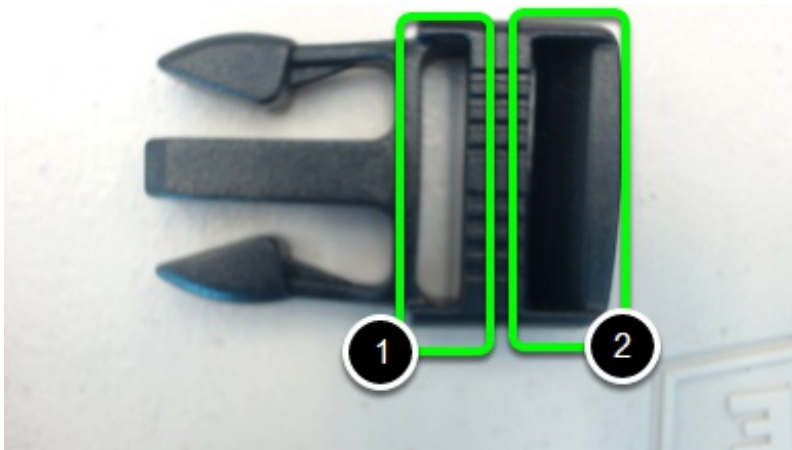

3. Once the belt is looped through the clip shown above the belt is ready to be buckled around the patients waist.

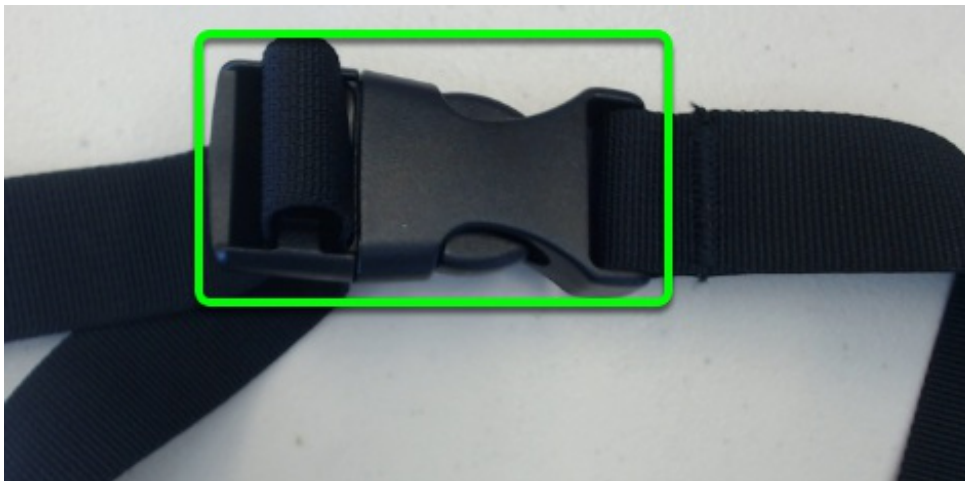

You can also watch the video below to see how to assemble your DynaROM belt.  
Your browser does not support HTML5 video.

