# How Do I Check My Software Version?

Last Modified on 05/19/2022 1:39 pm PDT

The MyoVision team routinely releases updates to the MyoVision software that include new <u>features</u>, bug fixes to old features, corrections to typos, and fixes to frequently occurring software errors. It's important to keep your software version up to date so you are getting access to everything your MyoVision system has to offer and so you're patients are getting the data from their exams in the most efficient and informative way possible.

### What Version Am I Running?

To check what version of the software you're running, open the "Help" menu then select the "About the MyoVision software" option.



When the Window opens the version number will be listed at the top. The most recent version of the software is **5.0.203** 

About		×
Installed Media: <b>MyoVision</b> Region and Language: <b>Default</b>		
MVSW43.EXE: <b>4,0,3,274</b> PBIDeviceComm.DLL: <b>1,0,0,11</b> VisionPort3G.DLL: <b>2,0,2,18</b> MyoTytron.DLL: <b>1,0,0,2</b>	L	
Feature Pack F : Drag and Drop Feature Pack G : Text to Mobile	ctivated hancements : Activated Graphic : Activated M and FlexUltra(tm): Activated p : Activated	1
My	oVision Atlas <sup>(m)</sup>	
	ickson PL NE #200 Seattle, WA 98125	
Copyright (C) 1998 - 2020	Tell me about Feature Packs	OK

# Which Feature Pack Do I Have?

On the Getting Started page of the MyoVision software, the program will tell you which Feature Pack you have activated. Each Registration Key is tied to a specific Feature Pack. If you are running an older Feature Pack you may want to consider purchasing a Software Annual Subscription package to receive all of the most recent Feature Packs.

#### The most recent Feature Pack is Feature Pack H

To see what is included in each Feature Pack look at the "About the MyoVision Software" window in the "Help" menu



## How Do I Get The Most Recent Software Release?

Visit myovison.com/download then click the Download Now button to download the MyoVision installer and updater \*.exe file.

When the file runs it will prompt you to select whether this is your first time installing MyoVision or whether you are installing an update. If you have previously installed MyoVision on this computer you will need to select the "I am installing an update..." option.

🍘 MyoVision Software - InstallAware Wizard		_	□ X	
Software Registration Key email reques Select the installation setup type that best s			<b>F</b>	
<ul> <li>Please select an item below, and chose "Next" to continue.</li> <li>I have a MyoVision Wirefree System, and would like to try all the new features including E-Scan Viral Marketing for 30 days prior to purchasing.</li> <li>I just purchased my Wirefree MyoVision system or software upgrade and have received my Software Registration Key instruction email.</li> </ul>				
○ I have already registered my software. I am installing an update at this time.				
NOTE: If you have not received a Software Registration Key and instructional email you are expecting, be sure and check your email Spam folder.				
PBI / MyoVision v 5.0.157	< Back	Next >	Cancel	

For further instructions on updating your software you can view our support article on to install a MyoVision update <u>Here</u>.