

# How to Find the Serial Number of Your Static Scanner Front-End

Last Modified on 08/31/2018 12:47 pm PDT


## Summary

This article will show you how to locate the serial number for the front-end piece of your ScanVision device.

## Locating the Serial Number

In order to get the serial number for the front-end piece of your ScanVision device, you will first need to remove it from the two piece body of the scanner.

1. Turn the Power Switch to off.

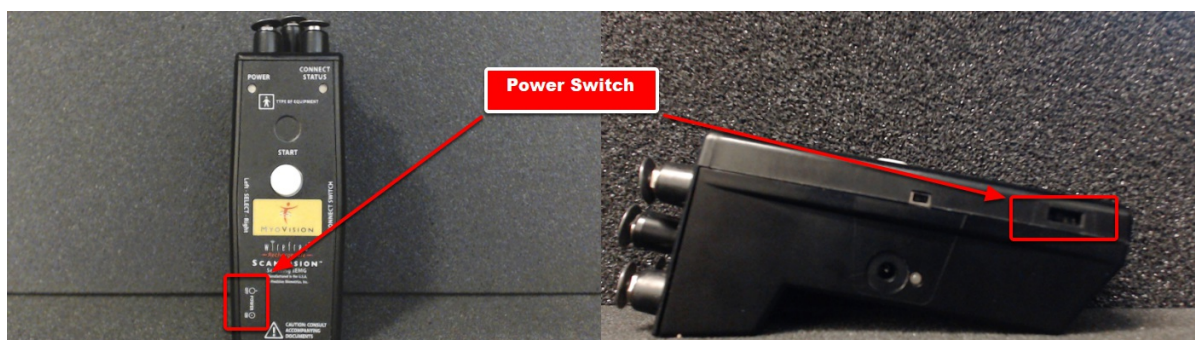


2. Remove the screw holding the front-end in place.





3. Pull the front-end piece of the body of the scanner,

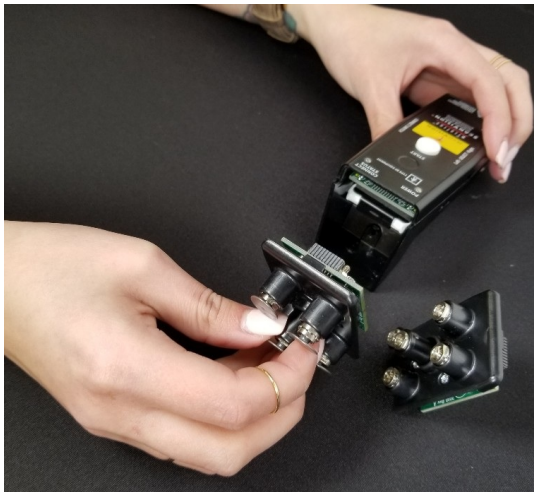

4. At this point you will be able to locate the serial number for the front-end piece of your static scanner.

The serial number will be on a white sticker located in one of two places.

- Directly visible on the board of the front-end. (Shown Below)



- The second location is under the board of the front-end. (Shown Below)

