

Troubleshooting Interpretation Report Templates


Last Modified on 07/10/2018 11:49 am PDT

Table of Contents

- [Summary](#)
 - [Check Templates Path](#)
 - [Edit Templates](#)
-

Summary


This article will show you what to do when your interpretation report templates aren't showing up.

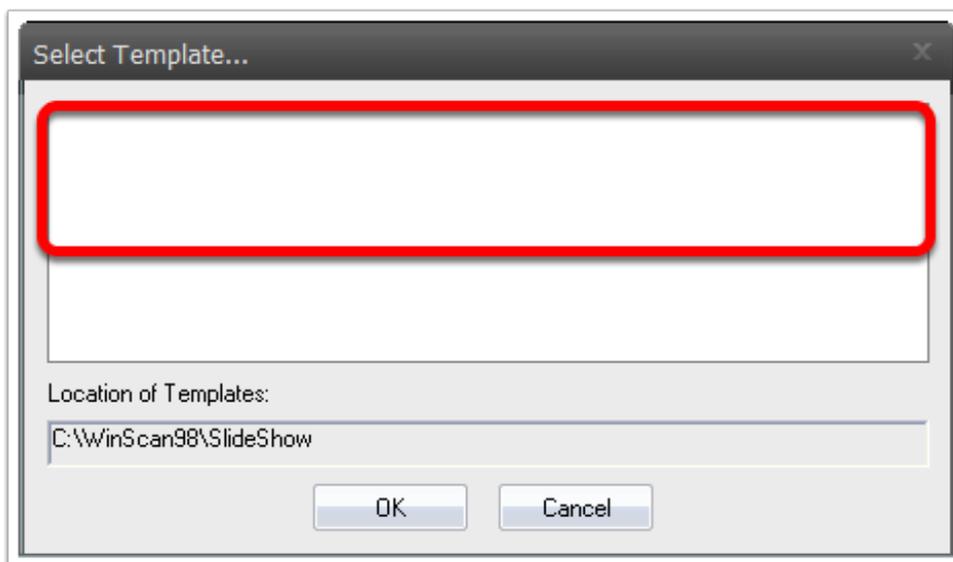

Check Templates Path


If your interpretation report templates aren't showing up the first thing to check is that the path for the interpretation report templates is correct.

1. Open the **Setup** menu from the top menu bar, go down to **Settings**, then go over to **Exam Interpretation**, and select **Local**.

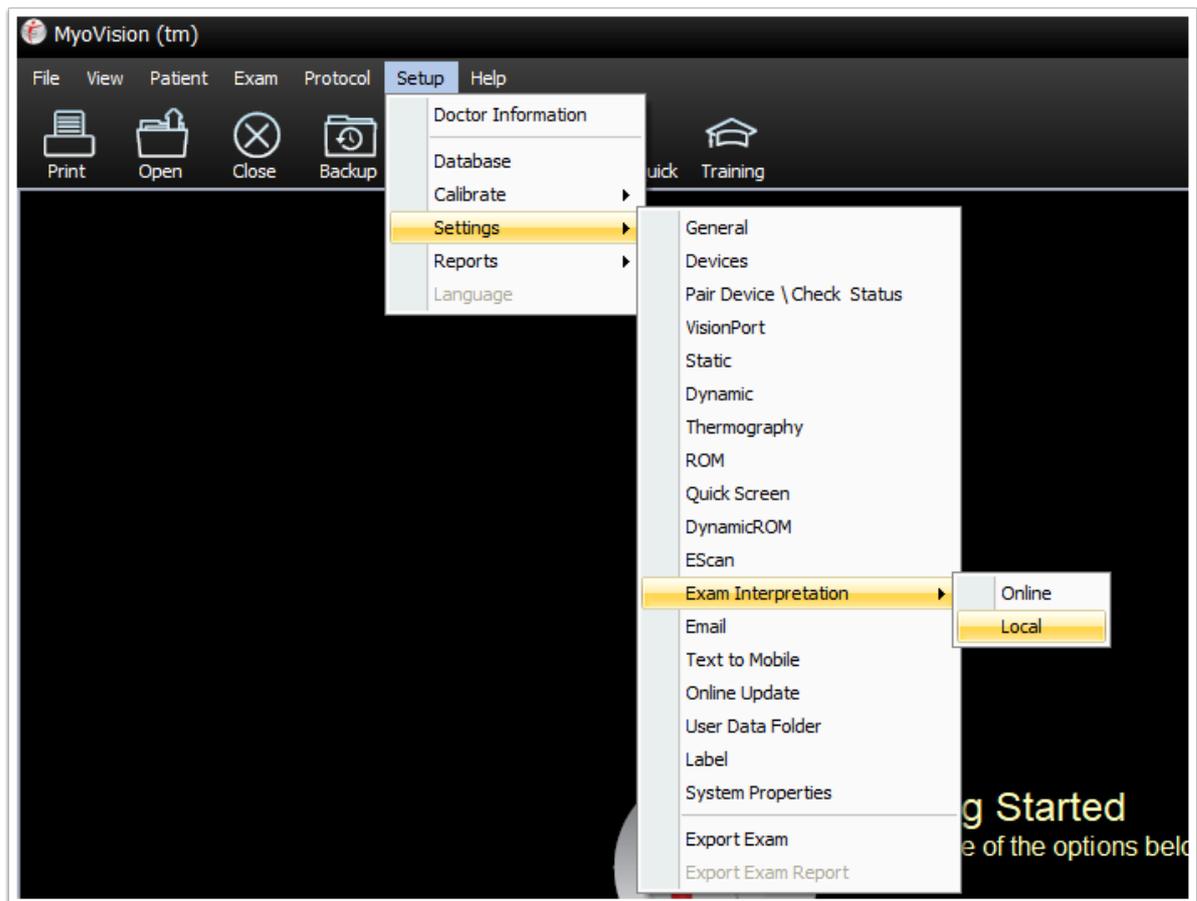


If this option is not available you are most likely running an older version of the software. Open the **Setup** menu, go down to **Reports**, and select **Interpretation**.


2. In the Interpretation Settings window look for the area marked Template. By default this should be set to C:\\Winscang8\\Template\\. To change the folder the Template path is set to click the button marked, "..."

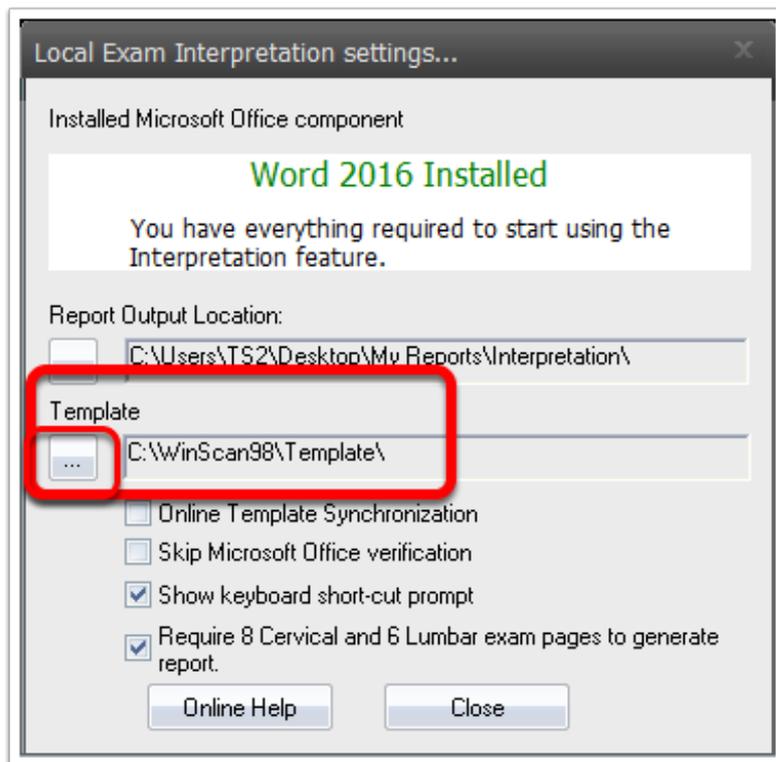

3. In the Browse for Folder window locate the C:\\WinScang8\\Template\\ folder and click OK.

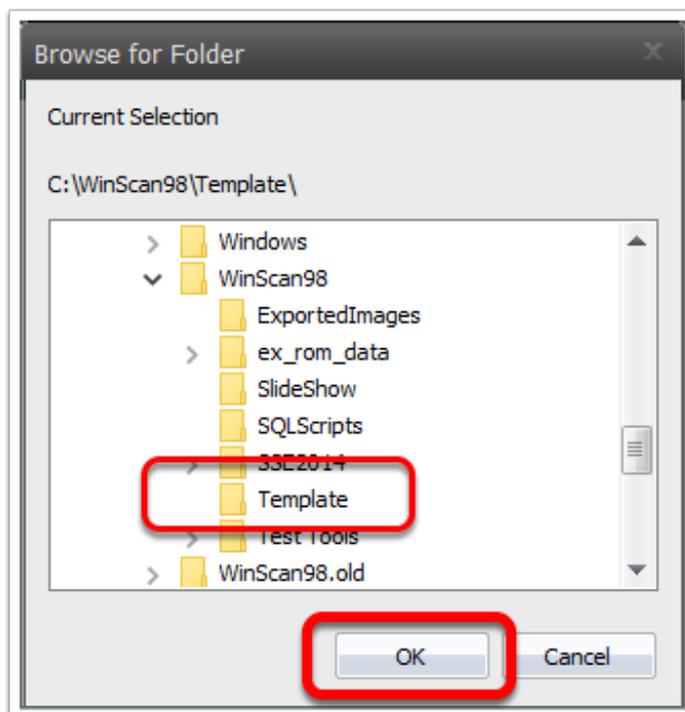

Click Close to save your setting change.

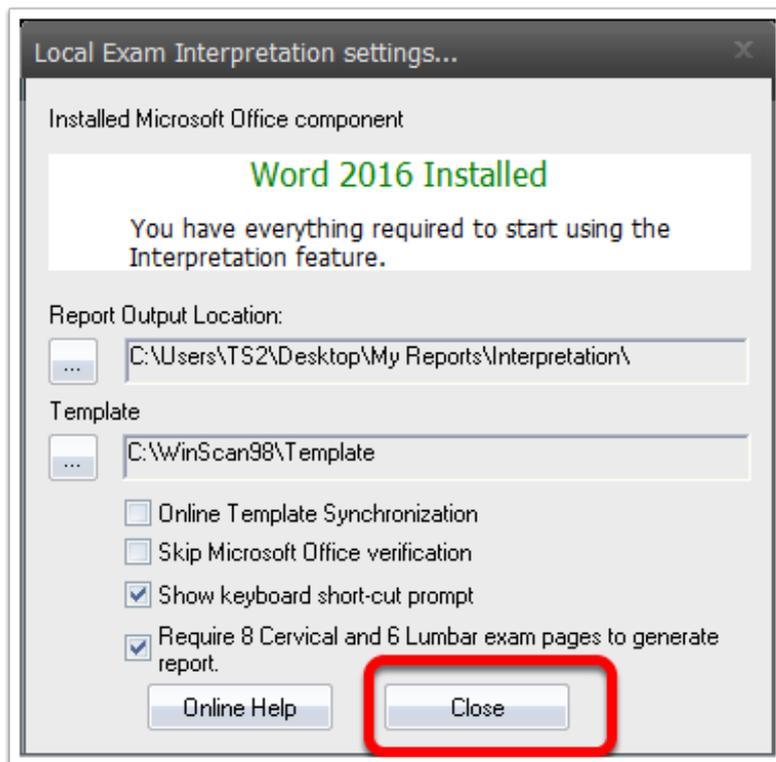

Edit Report Templates

You are able to edit and customize the templates used by the software for reports.

1. Open Windows File Explorer.
2. Then, navigate to C:\Winscang8\Template.

