


How Do I Download SQL Server 2014 Express?

Last Modified on 05/21/2019 11:36 am PDT

Summary

This guide will demonstrate how to check whether your computer is running a 32-bit or 64-bit version of Windows and download the correct version of SQL Server 2014 Express. [Click Here](#)

Open the search bar at the bottom of the screen (next to the Start button) and type "32-bit." Select the option for "See if you have a 32-bit or 64-bit version of Windows."

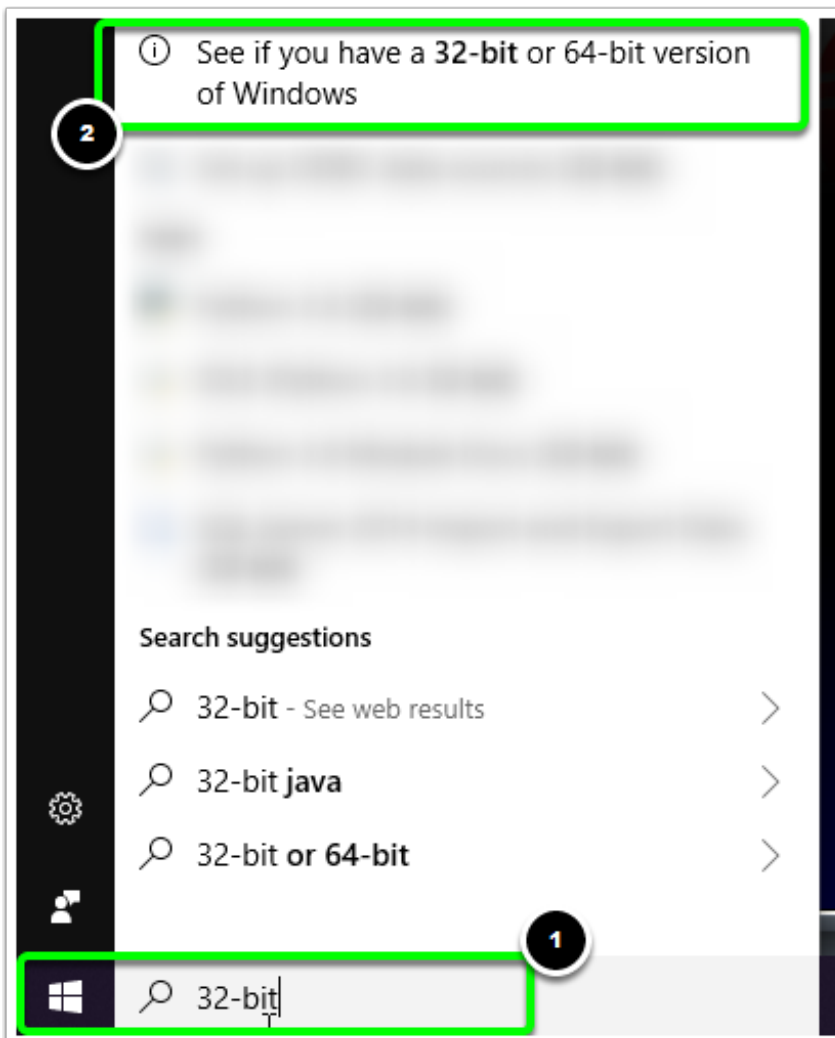

Next to "System Type" the Settings window will display whether your computer is running a 32-bit or 64-bit version.

Device specifications

XPS 8920

Device name [blurred]

Processor Intel(R) Core(TM) i7-7700 CPU @ 3.60GHz 3.60 GHz

Installed RAM 16.0 GB

Device ID [blurred]

Product ID [blurred]

System type 64-bit operating system, x64-based processor

Pen and touch No pen or touch input is available for this display

You will want to make note of whether your computer is a **32-bit** or **64-bit** operating system.

Next [Click Here](#) to open the web page where you can download SQL Server 2014 Express. Click the **"Download"** button and select the correct version of SQL Server 2014 for your operating system (**32-bit** or **64-bit**.) Click the **"Next"** button to start downloading SQL Server 2014 Express.

Microsoft® SQL Server® 2014 Express

Important! Selecting a language below will dynamically change the complete page content to that language.

Select Language:

File Name	Size
<input type="checkbox"/> Express 32BIT WoW64\SQLEXP32_x86_ENU.exe	149.9 MB
<input type="checkbox"/> Express 32BIT\SQLEXPXPR_x86_ENU.exe	168.4 MB
<input type="checkbox"/> Express 64BIT\SQLEXPXPR_x64_ENU.exe	196.7 MB
<input type="checkbox"/> ExpressAdv 32BIT\SQLEXPADV_x86_ENU.exe	1.1 GB
<input type="checkbox"/> ExpressAdv 64BIT\SQLEXPADV_x64_ENU.exe	1.1 GB
<input type="checkbox"/> ExpressAndTools 32BIT\SQLEXPRTW_x86_ENU.exe	840.8 MB

Download Summary:
KMBGGB
You have not selected any file(s) to download.

Total Size: 0

To begin the installation, right-click the "SQLEXPXPR" file and select "Open." For SQL 2014 Express installation instructions click [Here](#).

