


How Do I Make the MyoVision Software Text Bigger or Smaller?


Last Modified on 05/28/2019 3:30 pm PDT

The purpose of this article is to demonstrate how to customize the font size inside the exams to make it easier to read.

1. Click on "Setup" in the top menu, go down to "Reports" and select "Font"

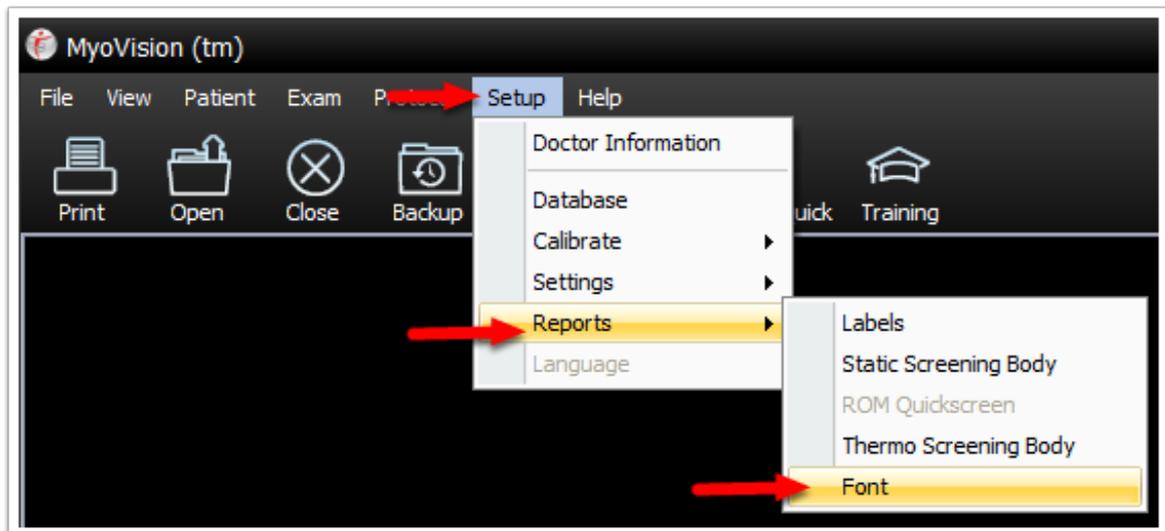


2. A window will open allowing you to make the following customizations: change the font, size, style, and color. Click OK and start a test exam to make sure you like the changes, and that the font doesn't cut off. For best results, the recommended font size is 9.

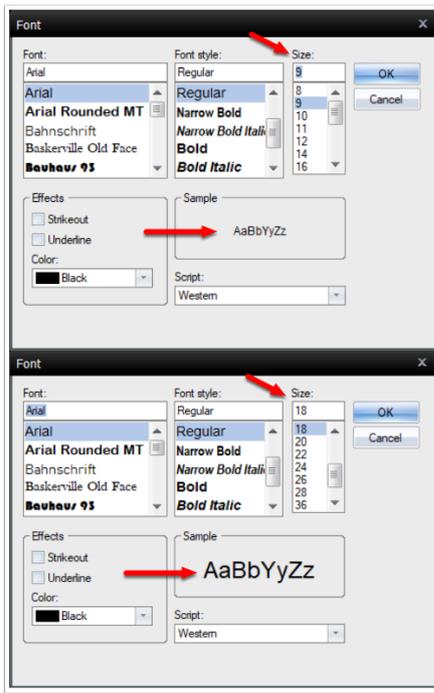

Below are two samples of the Font sizes.

A Sample Exam with the Font Size set to 9

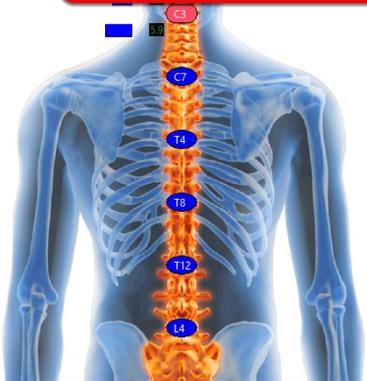

Static sEMG
Page 1 of 1
Position: STANDING

Sites
C2 C4 C6
T1 T3 T5 T7 T9 T11
L1 L3 L5

Next >

Patient Information:
Name:
Test:
ID Numbers:
-


Setting
 Autoscans
 Button Press Tone
 Site Announcement

Review Quit

Email Exam &/FS

ScanVisions Ready 22/30 sec.
 Skip any of the sites above by pressing the START button for 2 second.

A Sample Exam with the Font Size set to 18

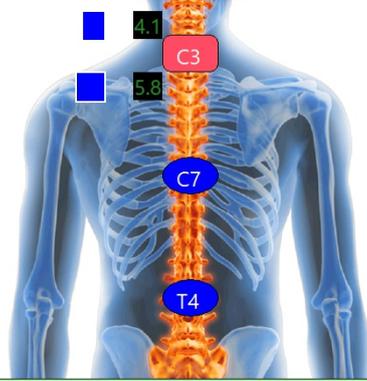

Static sEMG
Page 1 of 1
Position: STANDING

Sites
C2 C4 C6
T1 T3 T5 T7 T9
L1 L3 L5

ext >

Patient Information:
Name:

ScanVisions Ready 21/30 sec.
 Skip any of the sites above by pressing the START button for 2 second.

