

# Setting Up EScan with a Yahoo for Business Email Account

Last Modified on 01/19/2022 9:36 am PST

## Summary

This document shows you how to configure MyoVision for Yahoo Mail for Business


## Before you Begin

1. Make sure you are using Yahoo for Business, not plain Yahoo Mail as these require different settings.  
The guide for setting up Yahoo Mail is [here](#).

## How to Set Up Yahoo for Business


### 1. Open EScan's Email Settings

- Go to Setup > System > Email

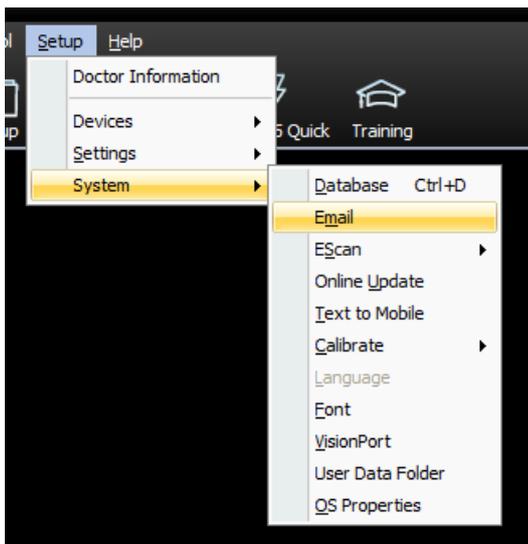


### 2. Change Host Name

- Click on Advanced.

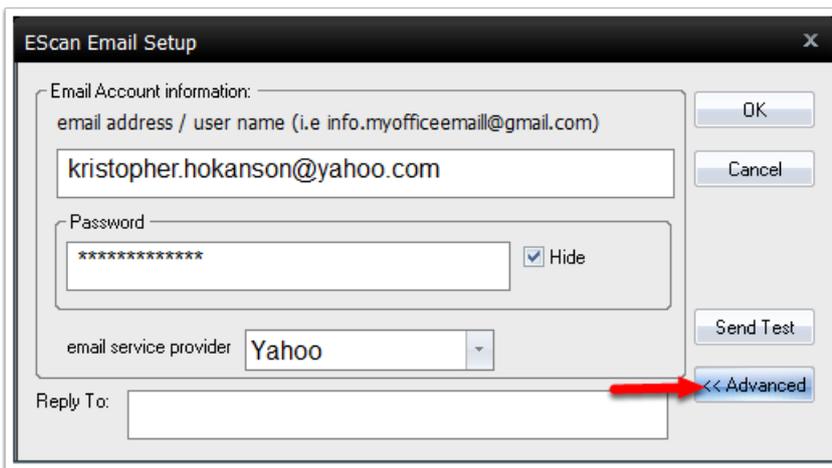

- Change the host name to "smtp.bizmail.yahoo.com" then click OK.

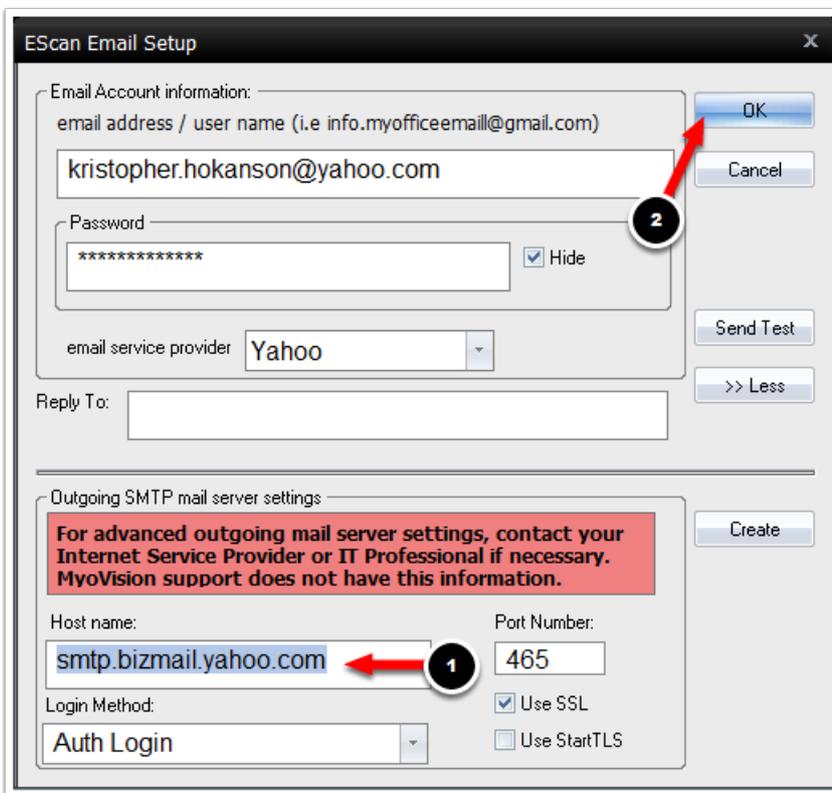

### 3. Create an App Password for Yahoo

- Follow the steps in the support article for setting up Yahoo Mail [here](#) to create an App Password. Once complete, Yahoo for Business will be set up to send email from MyoVision.

Note: If you need help at any point, please reach out to our technical support team at 1-800-969-6961, option 2. (International: 206-448-3464) Office hours are 8:00am - 4:00pm M-F, PST.