

How to Delete an Exam

Last Modified on 02/05/2020 3:14 pm PST

Summary


This document shows you how to delete a single exam (test) in the MyoVision software.

Before you Begin

1. Open MyoVision Software
2. **WARNING:** Once you delete an exam it cannot be undone and the data cannot be restored. Consider backing up your database before deleting the exam.
 - Not sure how to backup your database? Check out this support article: [Backup and Restore a Patient Database](#)
3. Make sure you have the latest version of the MyoVision software. Download here: <https://www.myovision.com/download.html>
 - [How Do I Check My Software Version?](#)
 - [Updating Your MyoVision Software](#)

How to Delete an Exam

1. Click Exam in the menu, then click Delete

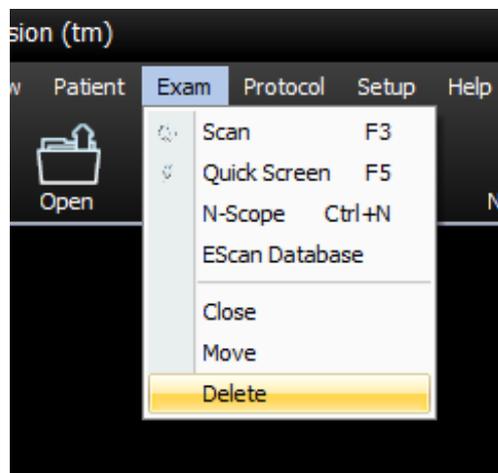


2. Find the patient with the exam you wish to delete


- **Tip:** Quickly type the first 3-4 letters of the patient's last name to quick jump to that portion of the patient list

3. Select the exam you want to delete, and click the Delete button

- **Note:** You can only delete one exam at a time

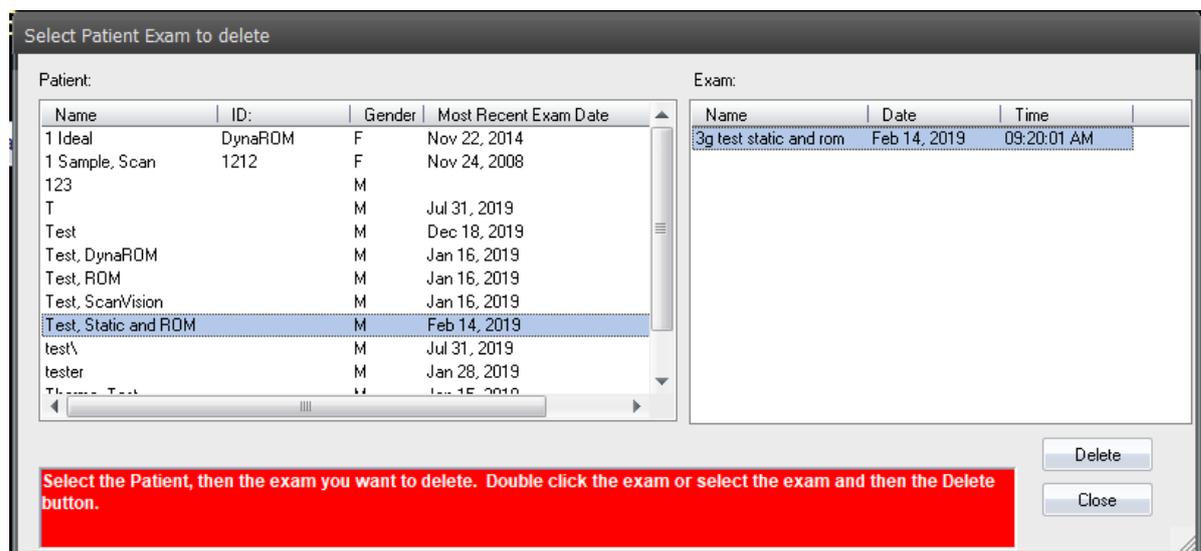

4. Confirm the deletion

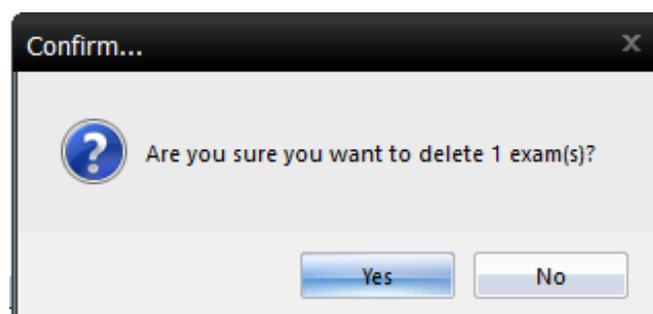

5. Done!

- Repeat steps 1-4 to delete another exam if desired
-