

How to Import Patient Files

Last Modified on 01/12/2021 6:25 pm PST

Summary

This document shows you how to import patient exam files (.wst) individually or to import all data files in a single folder

Need to Export Files? See our support article [How to Export Patient Files](#)

Before you Begin


1. **IMPORTANT:** Backup your Database prior to importing exams. It is possible to overwrite files which will result in loss of data
 - [Backup and Restore a Patient Database](#)
2. **IMPORTANT REMINDER:** Depending on country, state, or local laws you may be required to get patient consent prior to sharing or sending their data. **Please abide by these laws.**
3. **IMPORTANT:** Make sure **both** computers have the latest version of the MyoVision software. Download here: <https://www.myovision.com/download.html>
 - [How Do I Check My Software Version?](#)
 - [Updating Your MyoVision Software](#)
4. Have the files or folder accessible on the computer
5. Open MyoVision

How to Import Patient Exam Files

Import Specific and/or Individual Exam Files


1. Choose the option Import Individual Files

- Click on the drop-down menu "File"
- Choose "Import" then "Individual Files"
 - You can also press the control and I keys on your keyboard at the same time

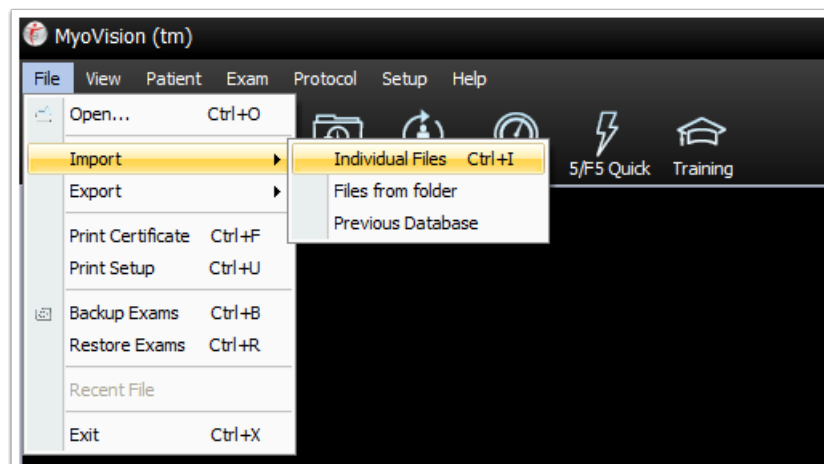


2. Navigate to the folder that contains the exams

- Click on the arrow for the drop-down box and choose the location the exams are saved in
 - This can be a folder or a folder on a USB drive, depending on where the exams are saved

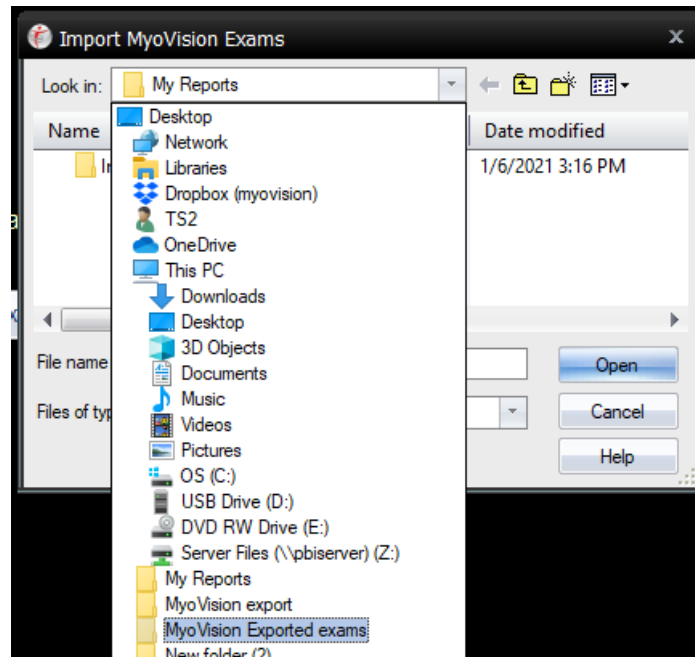


3. Choose the exam to import

- Click the exam to import then click Open

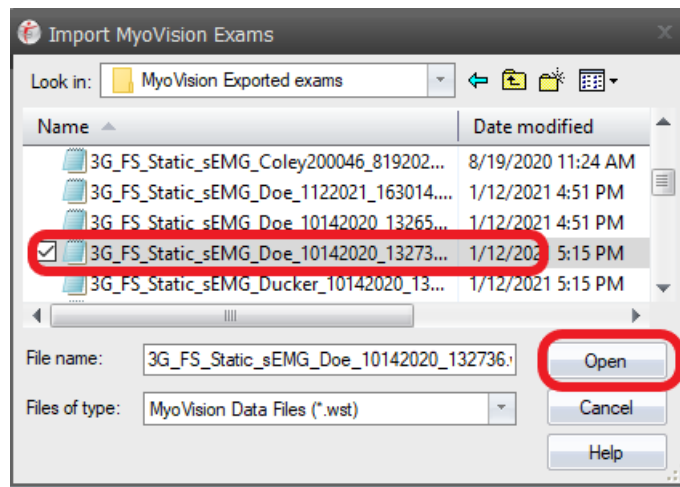

4. Confirm Import


- The software will confirm when the import is complete

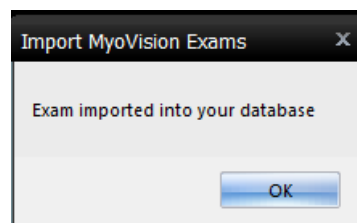

Import All Files in a Folder


1. Choose the option Import Files from Folder

- Click on the drop-down menu "File"
- Choose "Import" then "Files from Folder"

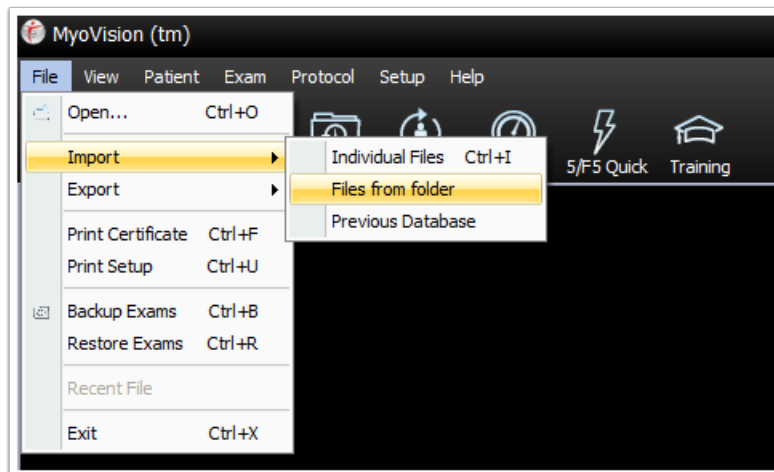

2. Navigate to the Folder

- Click on the folder that contains the files you want to import then click "OK"
- **NOTE:** If an exam is already found in the database it will not import and will show during the confirmation

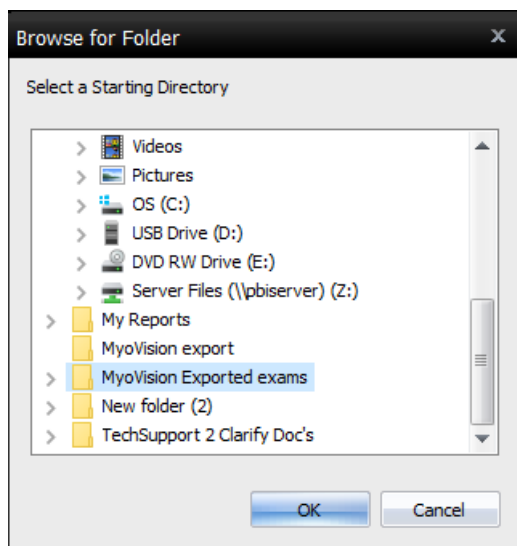

3. Confirm import

- The software will show how many exams were Imported from the chosen folder, if there are any errors, and if any exams are already in the database.

