

How do I Customize the Text Message for the Text to Mobile Escan Feature


Last Modified on 04/03/2024 7:40 am PDT

Summary

This document shows you how to customize the text message sent with the Text to Mobile Escan Feature

Before you Begin

1. Text-to-mobile is only available for customers with Feature Pack G or newer.
 - Not sure what Feature Pack you have? Open MyoVision and look in the lower right corner. If it says Feature Pack G then you have access to EScan.

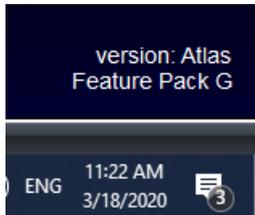


- Don't have Feature Pack G? You will need to [purchase a software subscription](#).
2. **IMPORTANT:** EScan must be setup to use the text-to-mobile feature.
 - See our support article [Setting Up EScan](#)
3. Make sure you have the latest version of the MyoVision software.
 - [How Do I Check My Software Version?](#)
 - [Updating Your MyoVision Software](#)


How to Customize the Text Message for Text to Mobile

Open "Text to Mobile" settings

- If not currently in an exam in the "Escan Information" window
 - Setup > System > Text to Mobile

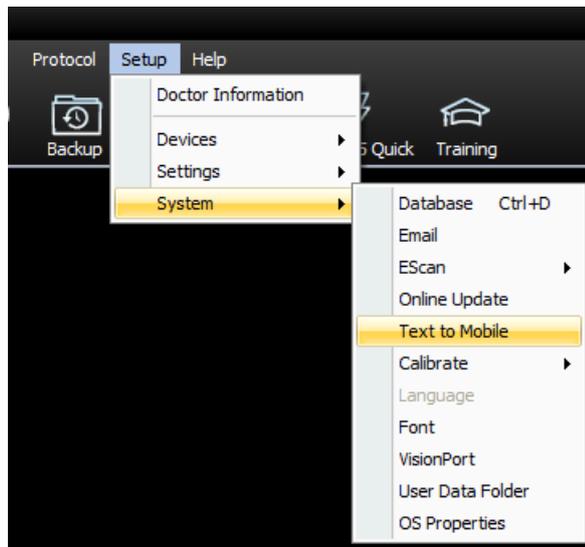


- If currently in an exam with the "EScan Information" window open
 - Click "click here" below "Text to Mobile Carrier:" drop down menu

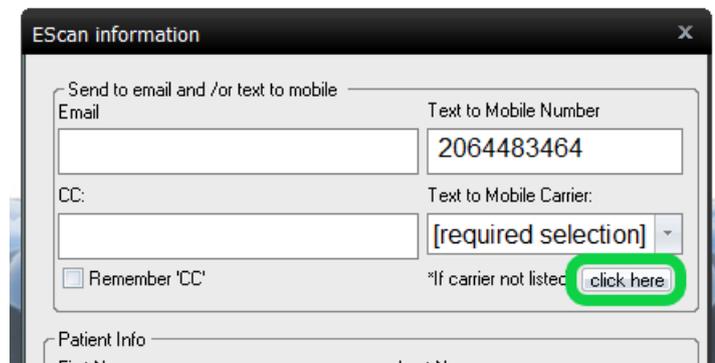

2. Enter your customizable message

- In the large text box under "Message" type your customizable message.
 - There is a maximum of 140 characters allowed.
 - Formatting (such as paragraph breaks) will show in the text message sent.
- Want to customize the image of the scan? See our support article [Customize the EScan Message](#)

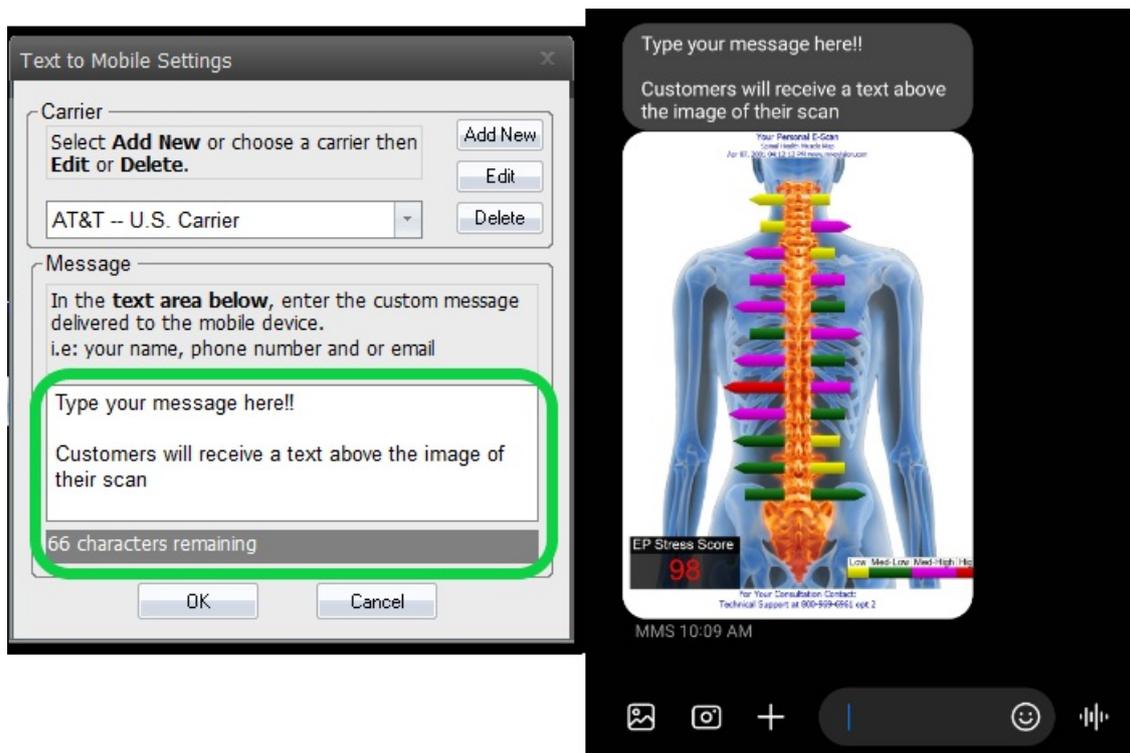

3. Close out of the Text to Mobile settings

- Click "OK" to save your Text to Mobile message
- Click "Cancel" to discard any changes made

