

How to remove ThermoVisionT2 early release software v1.12 or earlier


Last Modified on 09/06/2023 2:53 pm PDT

Summary

This document shows you how to remove the ThermoVisionT2 software for version **1.12 and earlier**.

Before you Begin

1. This is only for ThermoVisionT2 early release users **v1.12 and earlier**, and does not apply to the ThermoVisionT2 software v1.16 or later.
 1. If you have **v1.16 or later**, follow the steps in our installation article [HERE](#).
2. Settings will not be transferred during the update, there are two major software configurations to know prior to updating;
 1. Save location (if unsure, save an exam and look at the file path at the top)

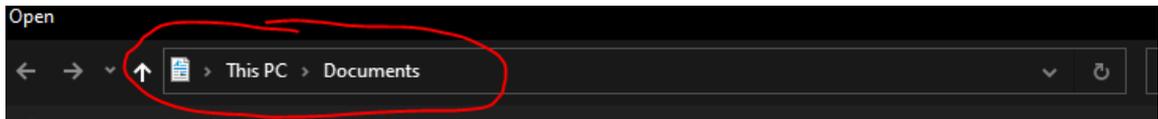

2. Minimum body temperature (found in Advanced Settings)

How to Remove ThermoVisionT2 software v1.08 or earlier

1. Remove the old version of the software

- Right-Click on the **ThermoVisionT2 vdx.x.x.x** Icon then choose **Delete**

2. Remove the Shortcuts, Pins, or Quick Access

- These are icons that allow easy access of the program but are not created by the software. If any were created, they will need to be manually updated when the software changes.
 - For Shortcuts  - Right-Click on the **Shortcut** then choose **Delete**
 - For Start Menu Pins  - See Microsofts support article [HERE](#)
 - For Taskbar Pins  - Right-Click on the **Taskbar Icon** then choose **Unpin from Taskbar**
 - For Quick Access  - See Microsofts support article [HERE](#)

Need to install the latest version of the ThermoVisionT2 software? See our support article [HERE](#) for link and instructions

